

PRECISION^{DX}™

Drug Testing Made Simple

The Precision DX Multi-Panel Saliva drug test offers rapid and inexpensive screening test for the simultaneous detection of drugs in human saliva.

- ◆ IMMEDIATE RESULTS
READS POSITIVES AT 5 MINUTES/ NEGATIVES WITHIN 2 MINUTES
- ◆ 24 MONTH SHELF LIFE FROM DATE OF MANUFACTURE
- ◆ SPECIAL PANEL CONFIGURATIONS AVAILABLE
- ◆ CHOOSE FROM 22 DIFFERENT DRUGS
- ◆ SIMPLE 2-PART DEVICE
- ◆ OBSERVED COLLECTION
- ◆ NO URINE HANDLING
- ◆ 99% ACCURACY
- ◆ CE MARKED

**DETECTS
DELTA-9
THC**



with color changing specimen collection indicator

Fast & Accurate Drug Testing Solutions

AmericanScreening

Ron Kilgarlin
318.798.3306 ext. 111
sales@americanscreeningcorp.com


PH: 318.798.3306
FX: 318.864.9900
TF: 1.866.526.2873

MI 2000.1 Rev B

PRECISION^{DX}™


Parameter	Calibrator	Cut-off	Parameter	Calibrator	Cut-off
Amphetamine (AMP)	D-Amphetamine	40	Ecstasy (MDMA)	3,4-Methylenedioxyamphetamine	50
Barbiturate (BAR)	Secobarbital	50	Mephedrone (MEP)	Mephedrone	100
Benzodiazepine (BZO)	Oxazepam	10	Heroin	6-Monoacetylmorphine	25
Buprenorphine (BUP)	Buprenorphine	5	Opiates (OPI)	Morphine	40, 25
Cocaine (COC)	Cocaine	30	Oxycodone (OXY)	Oxycodone	40
Cotinine (COT)	Cotinine	50	Phencyclidine (PCP)	Phencyclidine	10
EDDP (EDDP)	EDDP	20	Propoxyphene (PPX)	Propoxyphene	50
K2	JWH-018/JWH-073	50	Marijuana (THC)	11-nor- Δ^9 -THC-9-COOH	12
Ketamine (KET)	Ketamine	50	Marijuana (THC parent)	Δ^9 -THC	25
Methadone (MTD)	Methadone	30	Alcohol (ALC)	Alcohol	0.02%/0.04% 0.08%/0.30%
Methamphetamine (MET)	D-Methamphetamine	40			

INSTRUCT THE DONOR
TO **NOT** PLACE ANYTHING
10 MINUTES PRIOR TO COLLECTION.

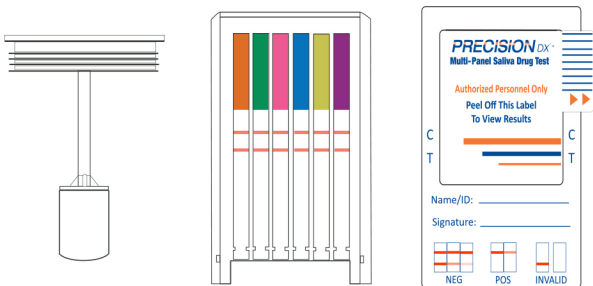

THE ORAL FLUID SPECIMEN SHOULD BE COLLECTED
USING THE COLLECTOR PROVIDED WITH THE KIT.

1 Collect Specimen

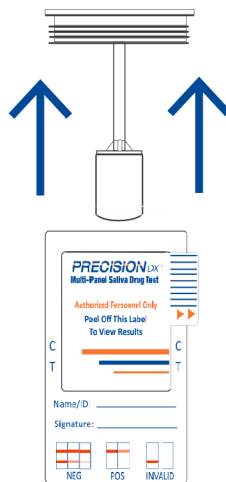


USING THE PROVIDED
COLLECTION SWAB.

SWAB THE INSIDE OF MOUTH
(CHEEK, GUMS, AND TONGUE)
FOR LEAST 3 MINUTES

UNTIL COLOR OF THE SATURATION
INDICATOR SHOWS COLOR


2 PLACE THE TEST ON A FLAT DRY SURFACE.

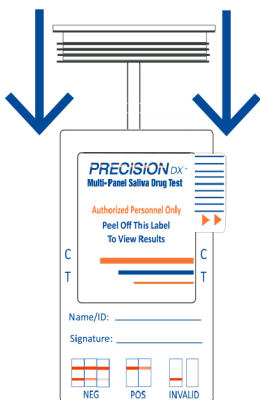





INSERT SPONGE FIRST,
INTO THE SCREENING DEVICE.

3 Read Results

WAIT FOR THE COLORED BANDS TO APPEAR

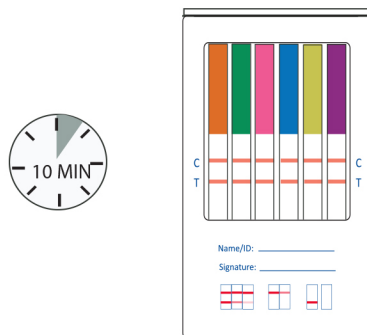

IN TEST RESULTS AREA.
READ RESULTS AT 10 MINUTES.

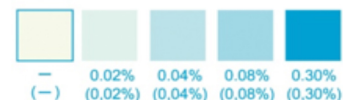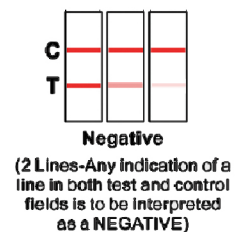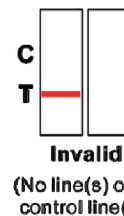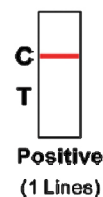

FOR ALCOHOL, READ RESULT AT 3MIN