


SalivaStik

Drug Testing Made Simple

The Precision SalivaStik offers rapid and inexpensive screening test for the simultaneous detection of drugs in human saliva.

- ◆ ONE STEP DRUG TESTING DEVICE
- ◆ UP TO A 24 MONTH SHELF LIFE FROM DATE OF MANUFACTURED
- ◆ ORAL SWAB STYLE DEVICE, NO SPITTING REQUIRED
- ◆ DRUG DETECTION LEVELS MEET INDUSTRY STANDARDS
- ◆ INTERPRETED RESULTS IN UNDER 5 MINUTES
- ◆ USER FRIENDLY AND NONINVASIVE
- ◆ CASE OF 25 QUANTITY
- ◆ GENDER NEUTRAL

**DETECTS
DELTA-9
THC**

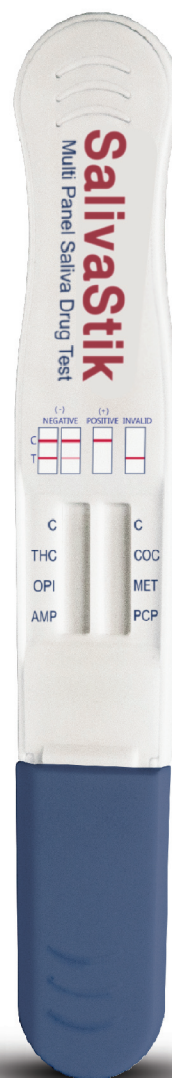

Fast & Accurate Drug Testing Solutions

AmericanScreening

sales@americanscreeningcorp.com

PH: 318.798.3306
FX: 318.798.3386
TF: 1.866.526.2873

ML25000.249 Rev.B

SalivaStik

Test	Cut-off(ng/mL)	Test	Cut-off(ng/mL)
Amphetamine (AMP)	50	Ketamine (KET)	50
Barbiturate (BAR)	50	Marijuana (THC)	12
Benzodiazepine (BZO)	10	Methadone (MTD)	30
Buprenorphine (BUP)	5	Methamphetamine (MET)	50
Cocaine (COC)	20	Opiates (OPI)	40
Cocaine (COC)	50	Opiates (OPI)	50
Cotinine (COT)	50	Oxycodone (OXY)	20
Ecstasy (MDMA)	50	Phencyclidine (PCP)	10
EDDP (EDDP)	20	Propoxyphene (PPX)	50
Fentanyl (FYL)	10		

*Customizable cutoffs available



1. Remove the blue cap by holding the sides and pulling gently. This will expose the collection pad. **Make sure there is a red line present in each window area indicating an unused device.**



2. Open mouth and gently rub the collection pad inside mouth against cheek in a circular motion several (approximately 15-20) times. Make sure to **keep head level**.



3. Gently rub the collection pad against the opposite cheek in circular motion several (approximately 15-20) times.




4. Gently rub the collection pad on top of the tongue several (approximately 15-20) times. **Do not chew, suck, bite or bend the collection pad.**



5. Rub the collection pad underneath the tongue several (approximately 15-20) times.

6. Place the collection pad underneath the tongue for approximately 30 seconds to collect saliva. Instruct the donor to hold the device in place with hand. **When sufficient amount of saliva is collected the blue lines will move. Repeat steps 2-6 until Red lines start to move upward**

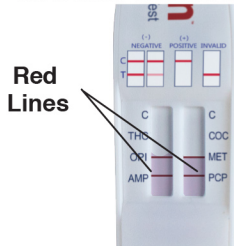

Test Windows


7. Remove from mouth as soon as blue lines move in both of the test windows. **Re-cap** the device.



8. Lay the device on a flat surface and read results in approximately 5 minutes after removing the device from mouth. Read package insert.

Inconclusive Result

A colored band at the **CONTROL (C) region** should be observed. When there is no colored band or shadow band at the specific **TEST region**, the test is inconclusive for that particular drug.

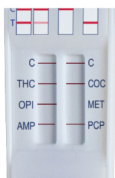

Example Interpretation:

MET: **Inconclusive**
THC: Negative
COC: Negative
AMP: Negative
OPI: Negative
PCP: Negative


Negative Result

For each test, two colored bands should be observed:

One in the CONTROL (C) region

One in the specific TEST region

The color of the test band may be slightly darker or lighter than the control band. Any visible band that can be seen is a **negative** result.

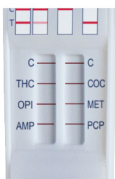

Example Interpretation:

MET: Negative
THC: Negative
COC: Negative
AMP: Negative
OPI: Negative
PCP: Negative

Invalid Result

When **no** colored band appears in the **CONTROL (C) region**, the test is **invalid** even if there is a band in the test region. **Repeat** the test with a new device.



Example Interpretation:

MET: Invalid
THC: Invalid
COC: Invalid
AMP: Invalid
OPI: Invalid
PCP: Invalid