

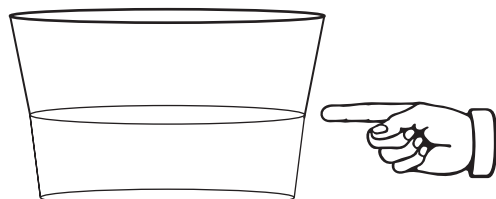
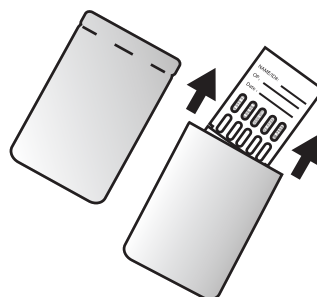
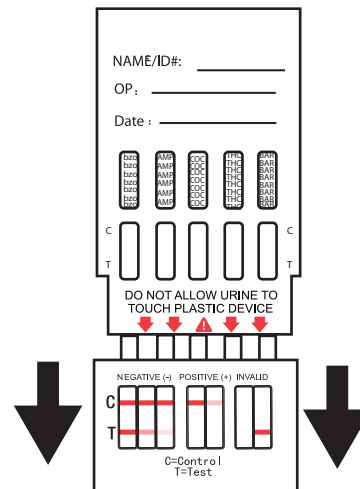
MULTI -PANEL DRUG TEST DIP CARD PROCEDURE SHEET

REVIEW INSTRUCTIONS.

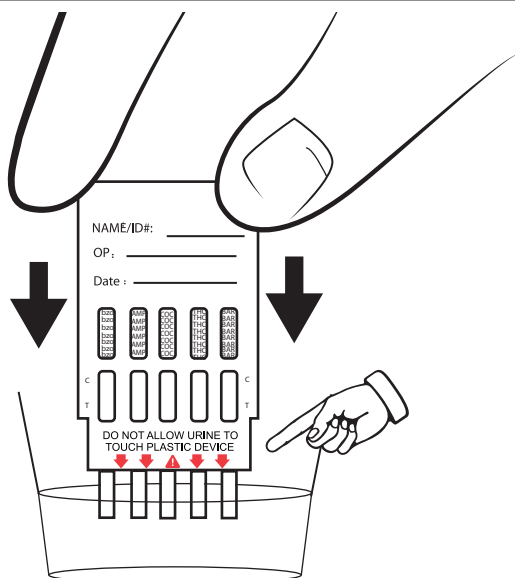
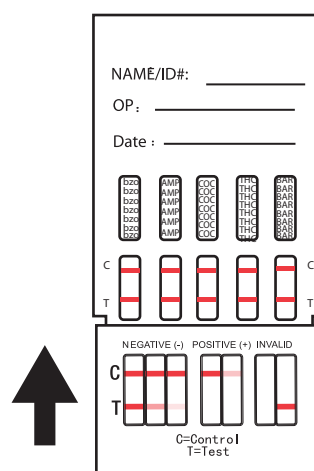
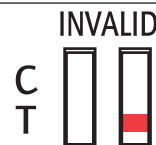
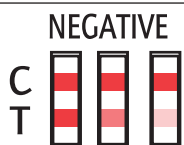
TEST DEVICE, PATIENT'S SAMPLE, AND CONTROLS SHOULD BE BROUGHT TO ROOM TEMPERATURE 68° - 86°F (20° - 30°) PRIOR TO TESTING.

DO NOT OPEN POUCH UNTIL READY TO PERFORM THE ASSAY.

DOUBLE CHECK THE EXPIRATION DATE. DO NOT USE IF THE PRODUCT IS EXPIRED.

**STEP 1**
COLLECT SPECIMENCOLLECT URINE SPECIMEN IN CLEAN CONTAINER.
RECOMMENDED LEVEL OF URINE IS 30 ML.**REMOVE FROM PACKAGE****STEP 2**
REMOVE CAP

REMOVE CAP FROM THE DIP CARD.

**STEP 3**
IMMERSE DIP CARDIMMERSE POINTED END OF TEST STRIP IN CONTAINER OF URINE.
DO NOT TOUCH PLASTIC WITH URINE.
LEAVE IN FOR 20-30 SECONDS.NEGATIVE RESULTS
WILL APPEAR ALMOST
IMMEDIATELY.
POSITIVE RESULTS AT FIVE MINUTES.**STEP 4**
REMOVE DIP CARDREMOVE DIP CARD AND REPLACE CAP.
PLACE FLAT ON DRY,
NON-ABSORBENT SURFACE AND WAIT FOR RESULTS.**POSITIVE:** A ROSE-PINK BAND IS VISIBLE IN EACH CONTROL ZONE (TOP BAND) WITH NO COLOR BAND APPEARING IN THE APPROPRIATE TEST ZONE (BOTTOM BAND).**NEGATIVE:** A ROSE-PINK BAND IS VISIBLE IN EACH CONTROL ZONE (TOP BAND) AND IN THE APPROPRIATE TEST ZONE (BOTTOM BAND). A FAINT, FADED, OR BROKEN BAND IS NEGATIVE.**INVALID:** A COLOR BAND IS NOT VISIBLE IN EACH OF THE CONTROL ZONES OR NO CONTROL BAND.

THIS IS A QUALITATIVE RESULT. WE RECOMMEND THAT IF YOU HAVE ANY QUESTIONS TO SEND IN FOR CONFIRMATION.

ML25000.27 REV B